Internal Characterization mit / with Tetric Color

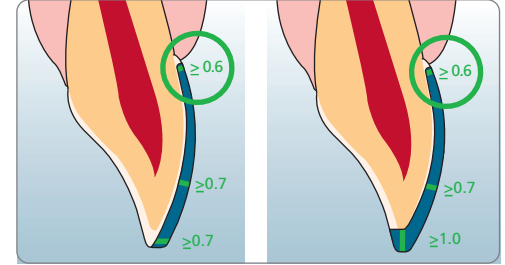
Before the treatment with IPS Empress CAD, determine the tooth shade to achieve the correct colour of the ceramic. Due to the translucency required for this patient, IPS Empress CAD HT (High Translucency) blocks in shade A1 were used.

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Following conservative treatment or the preparation of a mock-up, the shape data are exactly copied with the CORRELATION design program in the CEREC® 3D software. The surface can be recorded in detail if a contrast powder (e.g. IPS Contrast Spray Chairside) is applied.

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
The surface can be recorded in detail by applying IPS Contrast Spray Chairside. The dimensions and quality of the powdered preparation, surrounding gingiva and neighbouring teeth correspond to the initial record. The veneers are designed with the CORRELATION program. The QUADRANT program is used to fabricate several restorations on a virtual model.

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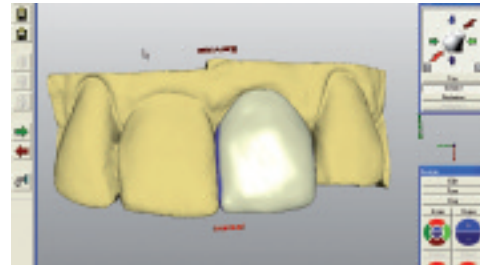By increasing the "Occlusal Offset" by 400-500 µm, the veneer is milled less. This increases the possibilities for contouring during finishing. The spacer and cementation joint remain unchanged as suggested by the system. Particularly smooth cementation surfaces and an even ceramic thickness can be achieved using the Endo Milling Mode.

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The restoration is tried in before the ceramic is etched and silanized. The final position and proximal contacts are checked. The exact fit is shown at the palatal-incisal interface.

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In preparation for adhesive cementation, the enamel is etched with phosphoric acid (e.g. Total Etch[®]). Subsequently, the dentin is thoroughly conditioned with, for example, Syntac[®].

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[®]ideally using a rubber dam / idealerweise unter Kofferdam

Parallel to the preparation of the teeth, the ceramic is conditioned with IPS Ceramic Etching Gel (60 sec). Once the gel has been carefully rinsed off with water, the bonding surface is silanized with Monobond-S for 60 sec.

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The dry silanized bonding surface of the veneer shows the typically mat surface, which ensures the sound micro-mechanical bond to the polymerized luting composite.

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Internal Characterization mit / with Tetric Color

Heliobond is applied to the rear side of the veneer and dispersed to a thin layer. This bonding layer serves as the base for the stains to be applied. This will ensure that the stains are evenly distributed and do not form any visible margins.

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First, a thin layer of light-yellow or ochre for characterization is applied to imitate the dentin core, which extends from the center of the veneer to the incisal third.

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A pronounced layer of ochre in the cervical and lower proximal area, and, if required, some medium- or dark-brown stains are applied. More intensive stains can be applied to the distal than to the mesial areas.

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Finally, the translucency of the incisal edge is reproduced using white or bluish-grey. The intensity of the stains applied* depends on the translucency and thickness of the ceramic used. The stains are polymerized with an LED curing light, e.g. bluephase®. Each segment is cured for 10 sec.

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The veneer is placed in the adhesive technique using a medium-viscosity, light-curing composite, such as Variolink® Veneer. The suitable brightness value of the composite will help adjust the brightness (value) of the veneer. Excess material is removed with a brush before polymerization. After light curing, the palatal areas are contoured.

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The margins are smoothed out, the labial surfaces are anatomically contoured and a natural texture is created. The slow reduction of the ceramic thickness will gradually bring out the stains placed under the ceramic. The shade effect in wet condition corresponds to that of the final polished situation.

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The surface is polished/smoothed out with low-abrasion rubber polishers (e.g. OpraFine) using water cooling and the reflective surfaces are selectively created. The final gloss is achieved with high-gloss brushes and diamond polishing paste.

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The completed IPS Empress CAD veneers harmoniously blend into the natural surroundings. A natural surface gloss is achieved. The incisal enamel is effectively reproduced with the true-to-nature translucency of the ceramic.

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